

# UNIVERSAL OPERATING TABLE

# SPL-331




**MIZUHO**



UNIVERSAL OPERATING TABLE

# SPL-331

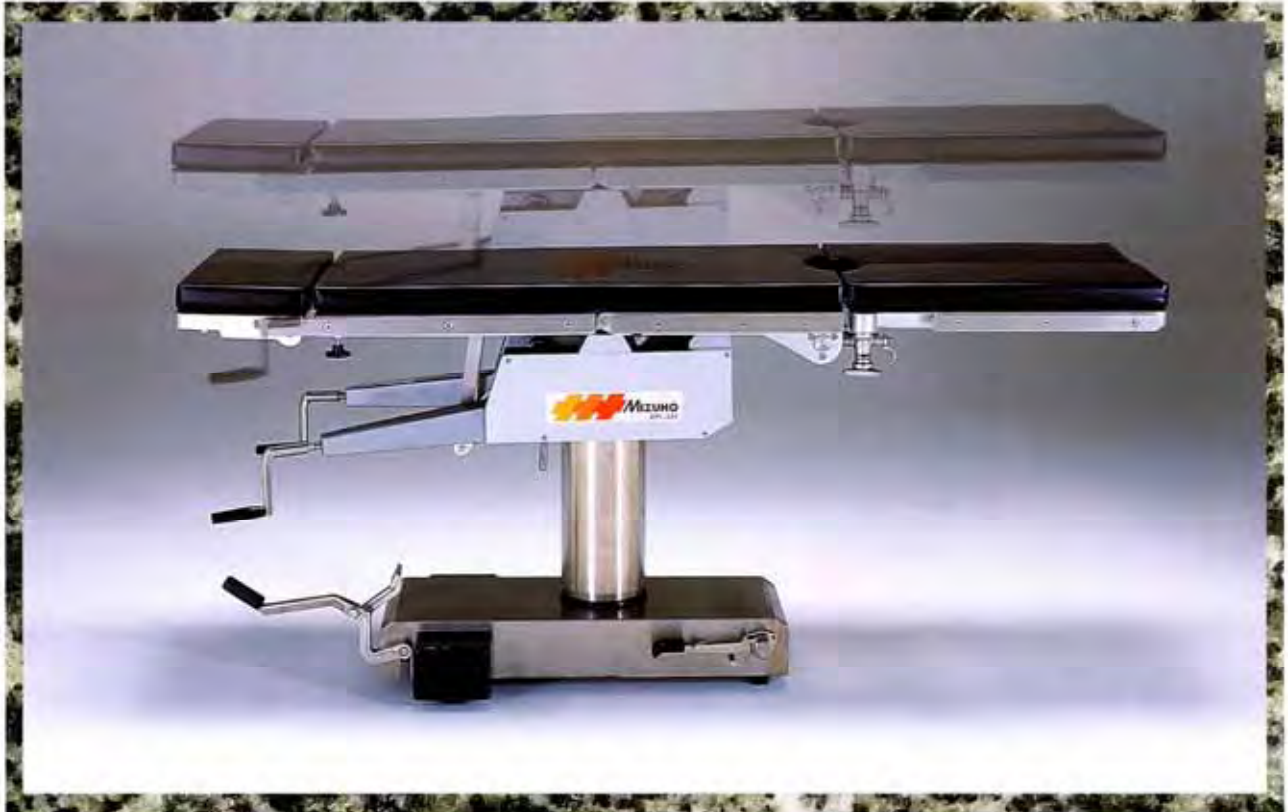

*This operating table is a product of Mizuho's clear-cut design concept to provide the users with an operating table possessing all basic functions needed for general surgery **plus the radiographic capability** with simple construction for minimized trouble and ease of maintenance, and above all an outstanding cost performance.*

## **Features**

Neatly encased head-end table control system along with the smooth and powerful foot-operated oil pump provides good maneuverability of the table as well as ease of maintenance.

Sturdily structured gear-driven tabletop control system ensures steady articulation of the tabletop for a variety of patient posturing free of wobbling and rattling and endures long service with minimized trouble.

Tabletop is designed to be rotatable by 180 degrees each clockwise or counter clockwise.

Availability of various accessories gives the table a added versatility.

Heavy-duty castor allows easy and smooth motion of the table in the operating room.

**MIZUHO**





● *Trendelenburg position*



● *Kidney position*

**Knurled knob for attaching/detaching the Head Section**

**Left-Handle**

Pulled-out position-  
Back-Section flexing  
Pushed-in position-  
Kidney positioning

**Right-Handle**

Pulled-out position-  
Trendelenburg positioning  
Pushed-in position-  
Lateral-tilt positioning

**Oil-Pump Pedal**

Tabletop elevating

**Lowering Pedal**

Tabletop lowering

**Leg Section flexing  
ratchet release**

**Knob for attaching/detaching  
the Leg Section**



● *Lateral-tilt position*



● *Urological position*



## UNIVERSAL OPERATING TABLE

# SPL-331

# Accessories



*Accessories for SPL331*

### Standard Items:

① Screen Flame	1 No.
② Arm Board with mattress	1 No.
④ Mattress	1 set
⑤ Shoulder Support with pat	2 Nos.
⑥ "Wakasugi" Body Support with pat	3 Nos.
⑦ Knee Crutches	2 Nos.
⑧ Foot Plate	1 set

### Optional Items:

③ "Wakasugi" Arm Rest	1 No.
M Type Face-down Head Support	1 No.(Not shown)
"Wakasugi" Flexible Wrist Strap	1 No.(Not shown)
Leg Strap	1 No.(Not shown)
"Wakasugi" Arm Board for hand surgery	1 No.(Not shown)
Auxiliary Radiographic Tabletop	1 Set(Not shown)
Douche Funnel with waste receptacle	1 No.(Not shown)

## Specification List of SPL331

### Functional specifications

Table Top Elevation	Highest	1010 mm
	Lowest	770 mm
Trendelenburg Angle	Normal	25°
	Reverse	25°
Lateral Tilt Angle	Right	20°
	Left	20°
Back Section Flexing Angle	Up	90°
	Down	15°
Head Section Flexing Angle	Up	15° (Head section detachable)
	Down	90°
Leg Section Flexing Angle	Down	90° (Head section detachable)
Leg Section Opening Angle		90°
Table Top Flexing		V-shape and Inverted V-shape
Table Top Rotation		180° each clockwise/anti- clockwise

### Weight & Dimensions

Tabletop Size	1930(L)x500mm(W)
Table Base Size	800(L)x310mm(W)
Weight	161 kg

## Mizuho Medical Co., Ltd.

ICN building 5F 27-17,2-chome, Hongo, Bunkyo-ku,  
Tokyo, 113-0033 Japan  
Fax No.+81-3-3818-1705 Phone No.+81-3-3815-7101  
E-Mail : marketing@mizuhomedical.co.jp

Manufacturer

**MIZUHO IKAKOGYO CO., LTD.**  
Tokyo, Japan